

Registration Number ECN-001941
Revision Number 1st
Subject MS Windows Security Update (Windows10 LTSC 2019) for FGR CR CL Series

Date 2020/02/20
Department Medical Systems Business Division
Originator KOUETSU KIKKAWA
Edited by
Approved by MASAYUKI MURAKAMI

Affected Products
DR-ID300CL, DR-ID330CL, CR-IR363AWS, FDR-3000AWS, MV-SR657EG

Scope of Application

Content

Applicable To (Serial Nos. Etc.)

Design Change Reason

Other

Content

Evaluation Completed

Action

Not Required (information only)

Content

Time Frame

Content

Completion Deadline

N/A

Notes

Estimated Work Time

Implementation H

Completion H

Charge

Not Applicable

Affected Documents

No

Content

Description Of Change

Other

Content

Refer to "1. Purpose" of ECN.

Supply Of The Parts

No

When

Name/Code

Quantity

Return Of The Replaced Parts To FUJI

No

Special Tools / Instruments

No

Content

Details

Refer to "3. Procedures for Windows Update" of ECN.

(For US Only)C&R Assessment

The action does not meet the definition of Correction and/or Removal under part 806.

Remark

SIAS Published

WW

Content

MS Windows Security Update (Windows10 LTSC 2019) for FCR CR CL Series

1. Purpose

We have evaluated if FCR CR CL Series works successfully with the following KBs, which have been distributed by Microsoft.

KB4534273 (Jun, 2020)

2. Applicable scope

2.1. Model, Version

DR-ID300CL	V12.1.0003 or later
DR-ID330CL	V12.1 or later
CR-IR363AWS、FDR-3000AWS	V9.2 or later
MV-SR657EG	V3.4 or later (*Production switchover scheduled for March 2020)

2.2. OS

Windows10 LTSCC 2019


3. Procedures for Windows Update

3.1. Opening the Internet Explorer

Open the Internet Explorer to open Microsoft update Catalog (URL:
<https://www.catalog.update.microsoft.com/Home.aspx>).


3.2. Searching for the KB

In the [Search] box on the Microsoft Update Catalog page, enter “4534273”.
Click “Search”.

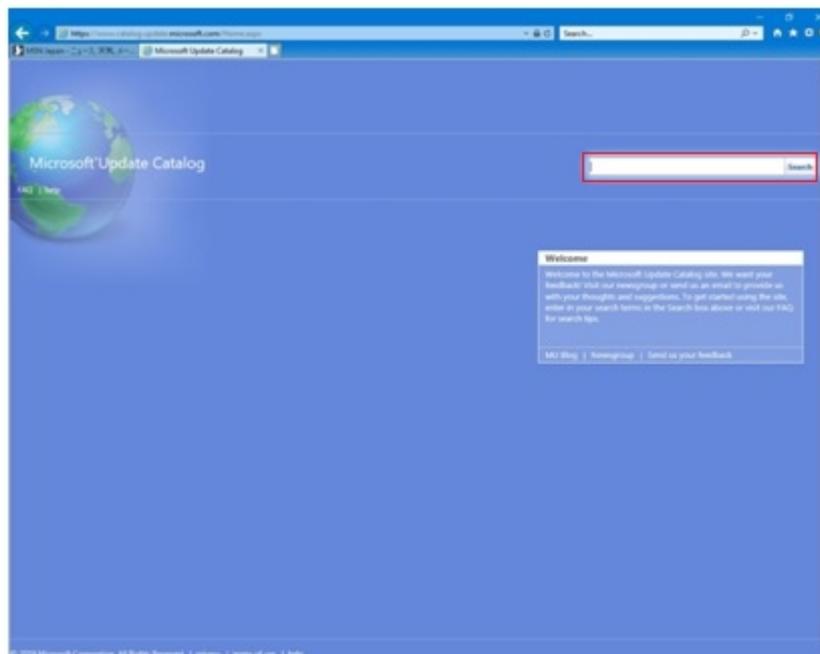


3.3. Downloading and installing the KB

- 1) “Search results for ...” appears. Click [Download] on the right side of “2020-01 Cumulative Update for Windows 10 Version 1809 for x64-based Systems (KB4534273)”.

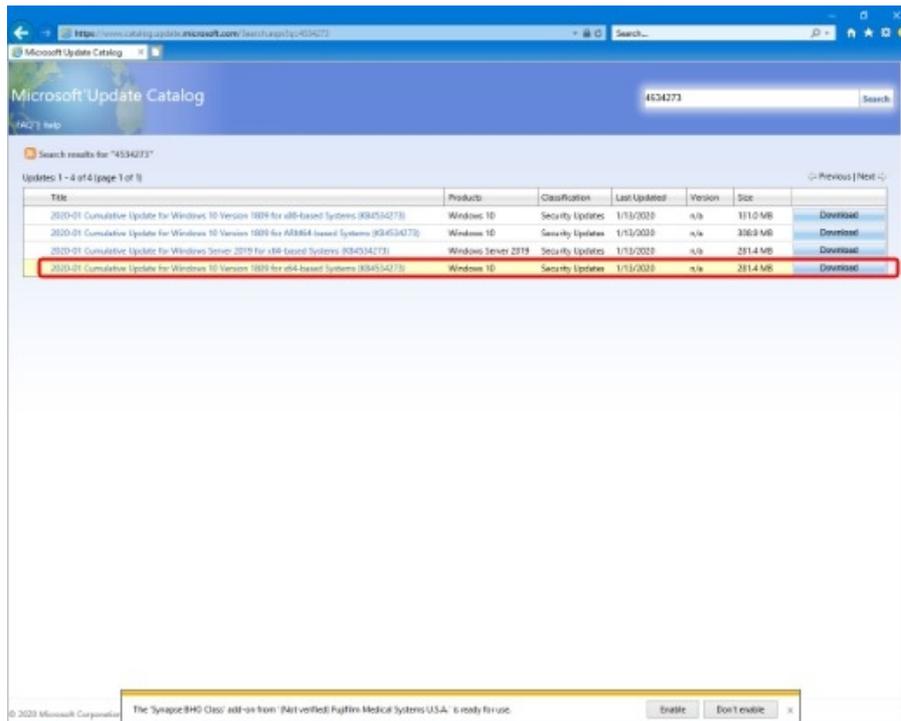

- 2) “Download” appears. Click “windows10.0-kb4534273-x64...”. Click [Open] at the bottom of the window.

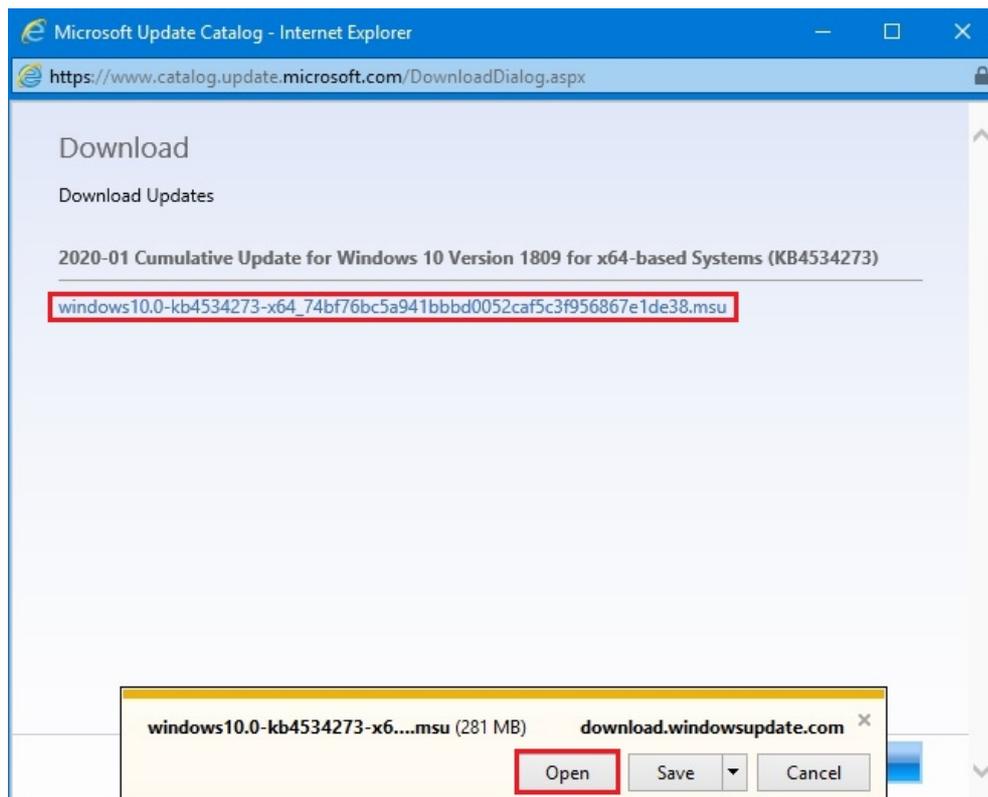

3) "Windows Update Standalone Installer" appears. Click [Yes].



4) When the installation finishes, "Installation complete" appears. Click [Restart Now].

* Clicking [Restart Now] restarts the PC.

