

TO: WMI Authorized Dealers

FROM: White Mountain Imaging, Technical Support

DATE: 04.24.2012

SUBJECT: Add Additional Nodes to Osirix

Product: Osirix

Symptom: Add Additional Node to Osirix

Resolution:

1. They can do this by going to Preferences, Locations and click on “Add New Node”
2. They can then enter the following information:
 - a. IP address of node
 - b. AE Title of node
 - c. Port Number of node
 - d. Be sure to check the Q&R and Send boxes
3. Make sure they save changes and “click the lock to prevent further changes”

