

Service Bulletin

TO: WMI Authorized Dealers

FROM: White Mountain Imaging, Technical Support

DATE: 10.09.2012

SUBJECT: FDX Upgrade to v6.2

Product: FDX Console

Symptom: Upgrade Instructions

Resolution:

1. Clear data base and Output Queue.
2. Go to FUJIFILM icon in the system tray, right click, click end (this will take up to 10 seconds).
3. Insert v6.2 disk
4. Select Option 5
5. Select Option 2
6. Reboot the app
7. Insert hot fix 1
8. Copy file in the c drive, unzip
9. Run setup.bat
10. Select option 1
11. Select option 0
12. Delete the file from the c drive
13. Replete for hot fix 2
14. Go to user utility parameter update batch update auto send
15. Reboot system

Check version should say v6.20003

