

TO: WMI Authorized Dealers

FROM: White Mountain Imaging, Technical Support


DATE: 10.16.2014

SUBJECT: Import Images to OmniVue

Product: OmniVue

Resolution:


1. Insert CD and browse to the images file
2. Copy image files (or drag and drop) to C:\Genesis Digital Imaging\OmniVue\Image

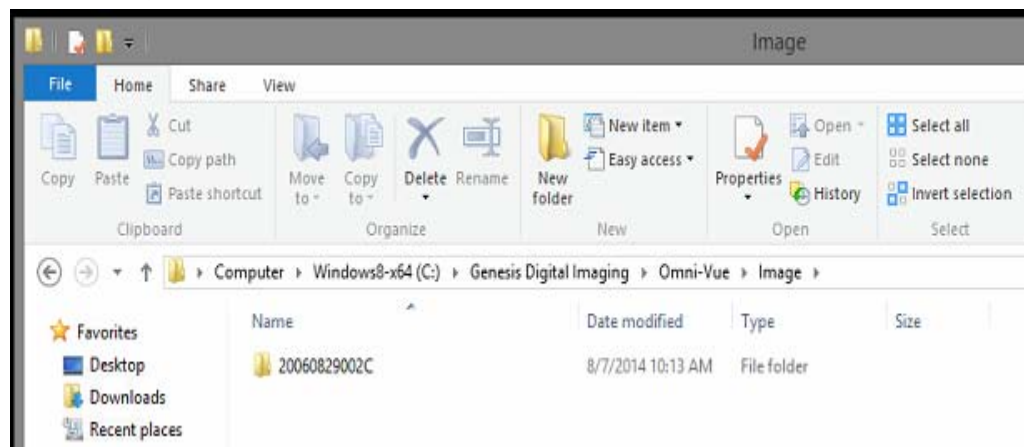


3. Launch OmniVue (password: genesis)




- 4. On the first screen click "Close" in the top right hand corner

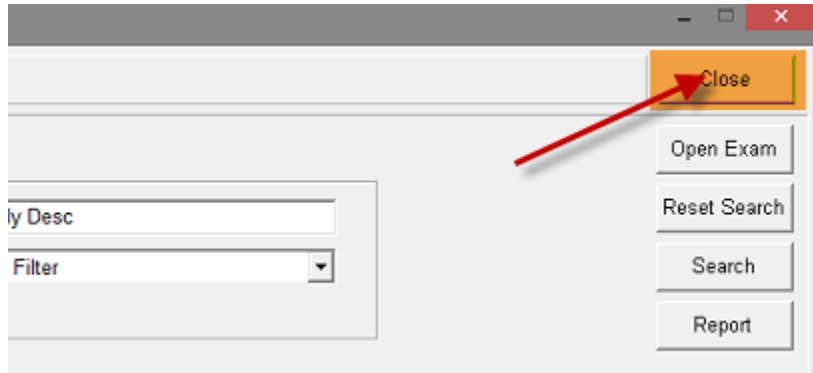

- 5. Then Click: File > Batch Import

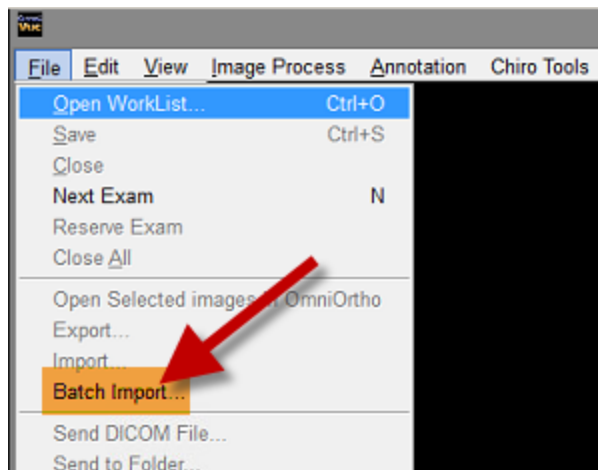

- 6. Check the two boxes and click "Start"

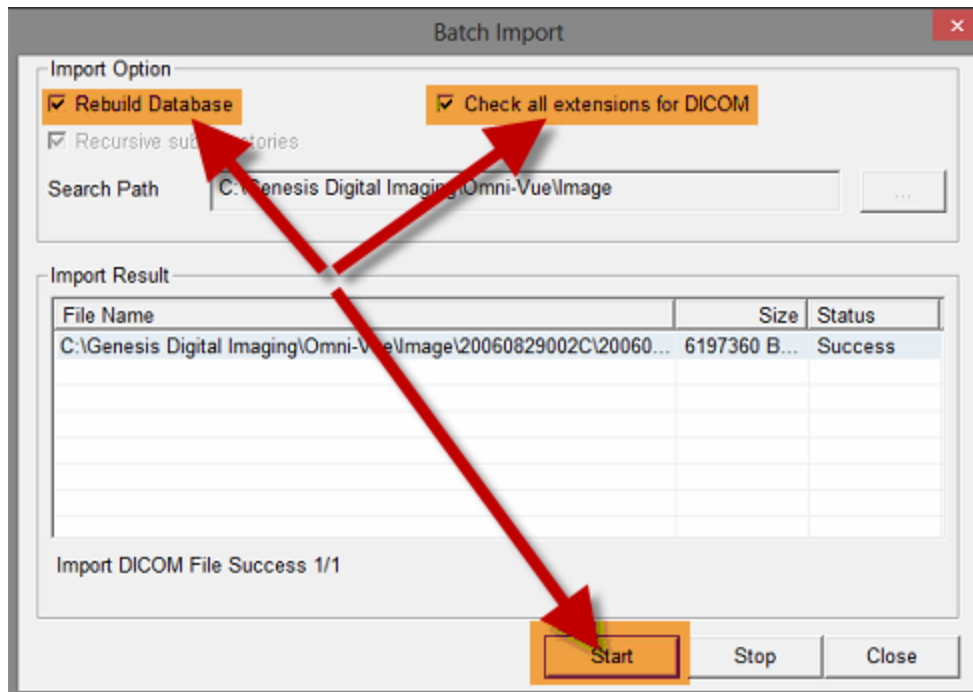

Service Bulletin

- 7. When it you see "Import DICOM File Success x/x" (x being the number of files) click "Close"
- 8. To launch the work list click on the file folder in the top left of the Tool List



- 9. To refresh the work list and see imported studies clear the check boxes for the date range and click "Search"

