


TO: WMI Authorized Dealers
FROM: White Mountain Imaging, Technical Support
DATE: 04.04.2017
SUBJECT: PACS PRO Viewer Registration


Product: PACS PRO Viewer

Resolution:


1. Install software
2. After software has been installed you will be prompted to register the product. Go to www.pacsregistration.com/tiba

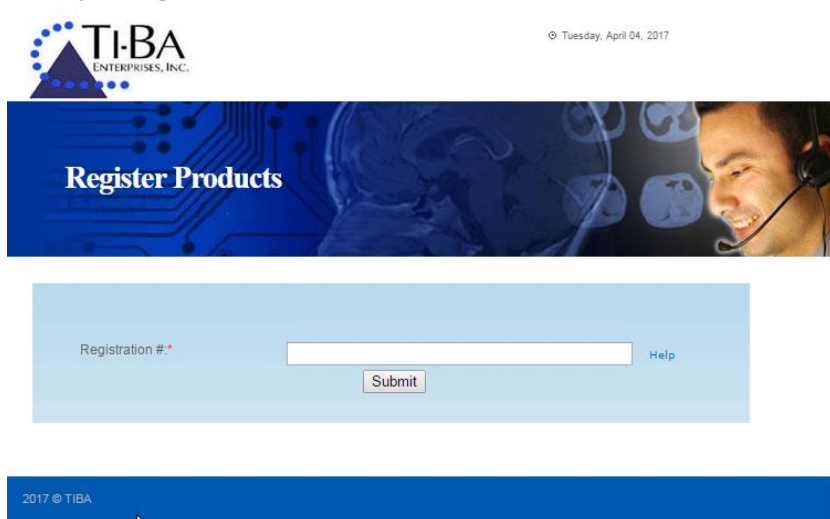

3. Enter registration number you received and click “Submit”
4. Enter in required information on next screen

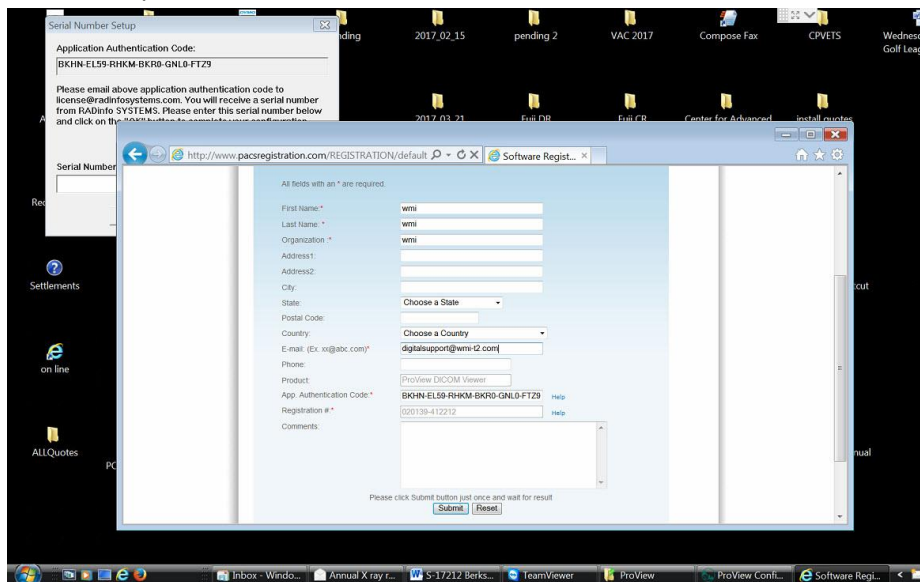

5. Application Authentication Code will be found on “Serial Number Setup” window
6. Click on “Submit”
7. A serial number will then be generated for you.
8. Copy and Paste this into the “Serial Number” field in the “Serial Number Setup” window

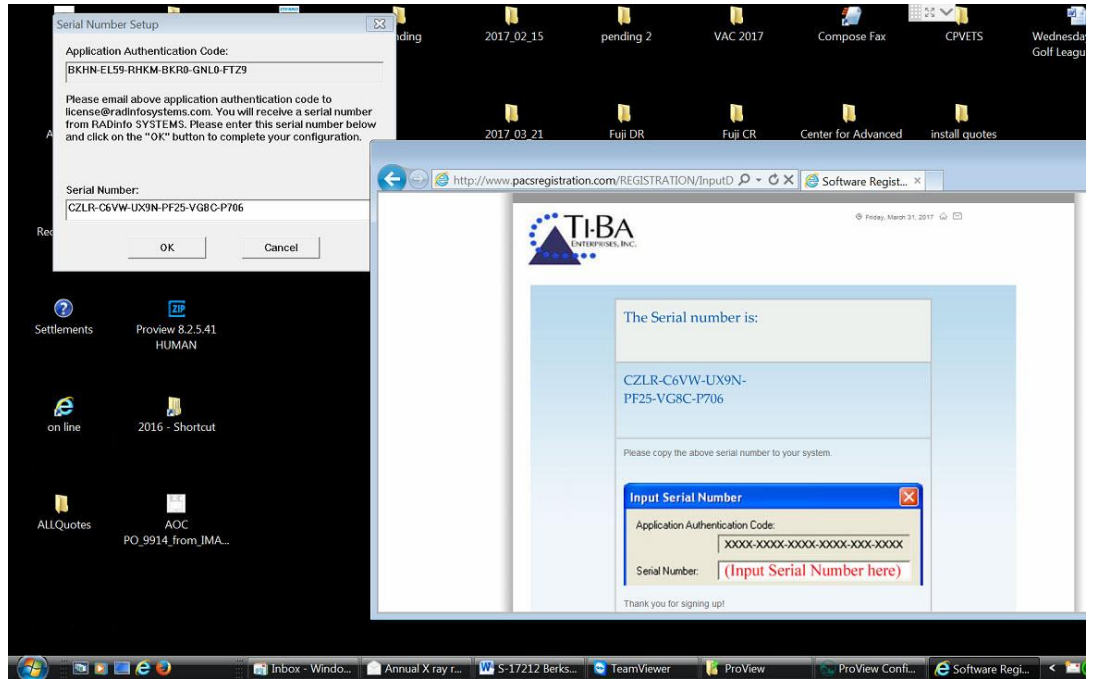

9. Click “OK”

