

TO: WMI Authorized Dealers

FROM: White Mountain Imaging, Technical Support

DATE: 9.29.2011

SUBJECT: Stitching for FCR Prima IIP

Product: FCR PRIMA IIP

Symptom: How to: Image Stitching


Resolution:

Stitching Images On the FCR Prima IIP

1. Register study and choose Exposure (ex: Whole Leg or Whole Spine)
2. Start study and run Imaging Plates
 - a. Because there is no Bar Code Reader you must be sure to run plates in the correct order




3. After last plate is complete click on first exposure and you will see the “image stitching” icon light up.
4. Click the “Image Stitching” icon

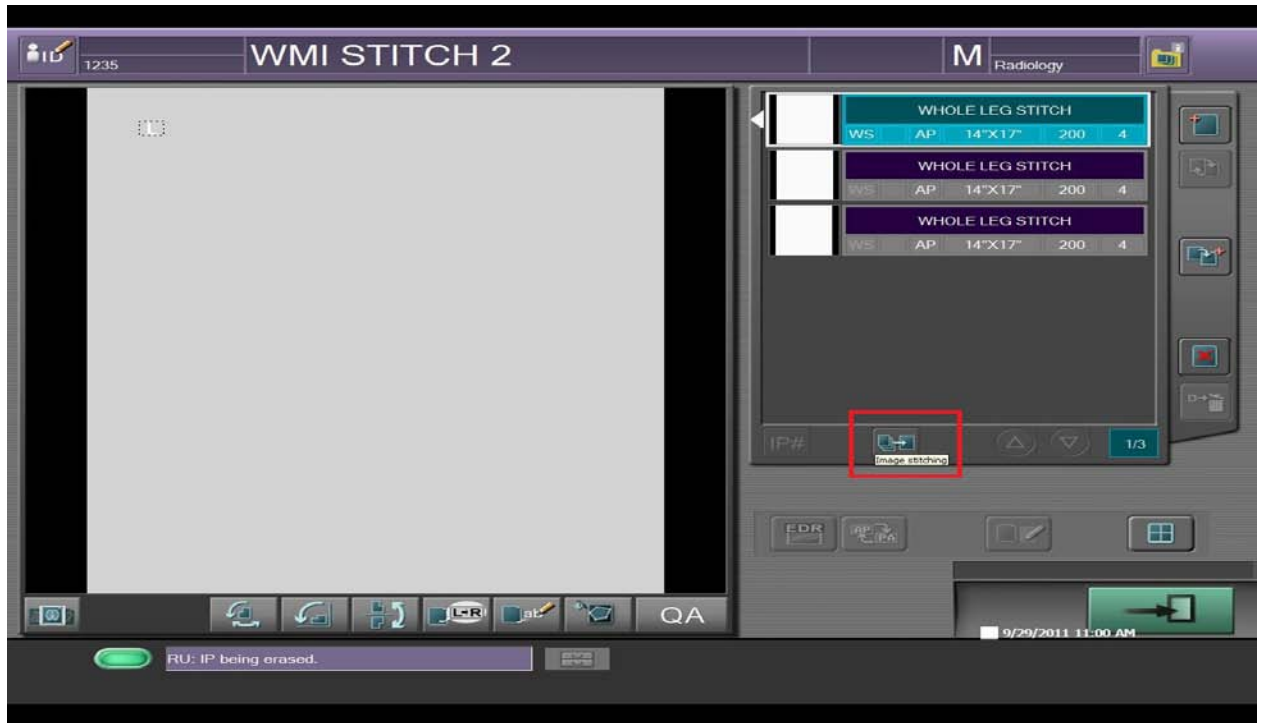

5. Click “Yes” when asked “Do you want to perform provisional processing” and the preview window will display.



Service Bulletin

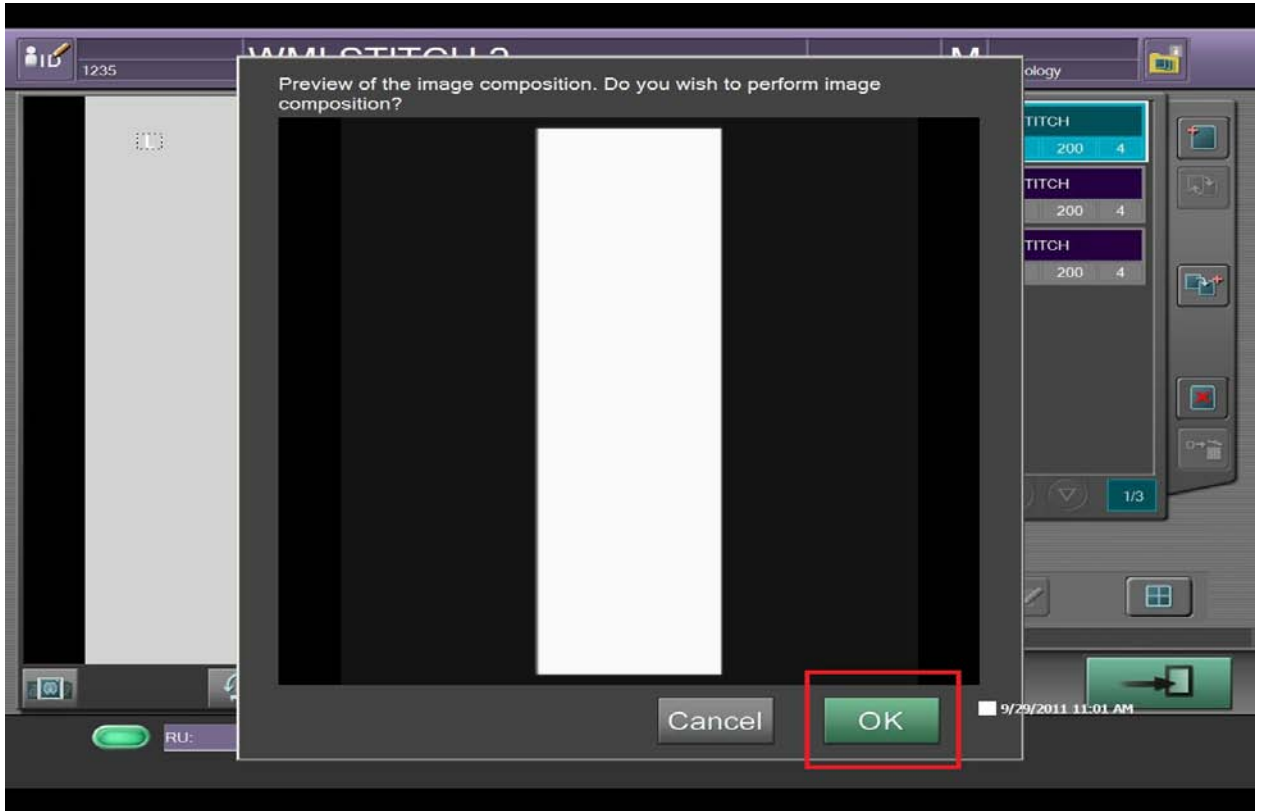

6. Click "OK" and your image will auto stitch.



7. Complete the study.

