

TO: WMI Authorized Dealers

FROM: White Mountain Imaging, Technical Support


DATE: 03.03.2014

SUBJECT: Print Image With Annotations to .PDF From OmniVue


Product: OmniVue DICOM Viewer

Resolution:


1. Download PDFCreator to pc and install.
 - a. <http://www.software-watcher.com/listing/122661/PDFCreator?did=10940&pid=1&ppd=search,38695423595,pdfcreator,e,,c,0,,,&gclid=CLLE7uyD97wCFaVxOgodLEcABw>
2. Open study to print in OmniVue. Click on File>Print

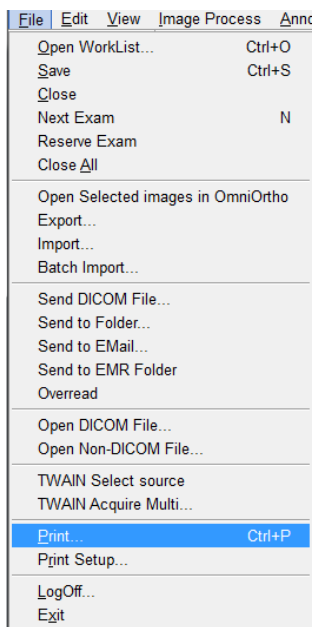

3. In the Print box choose "Paper"

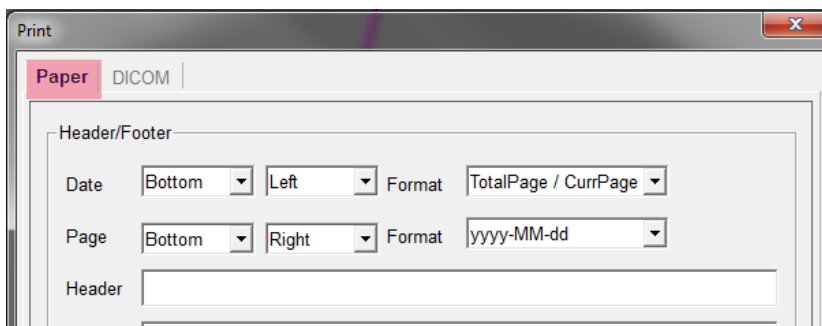


4. Choose print options

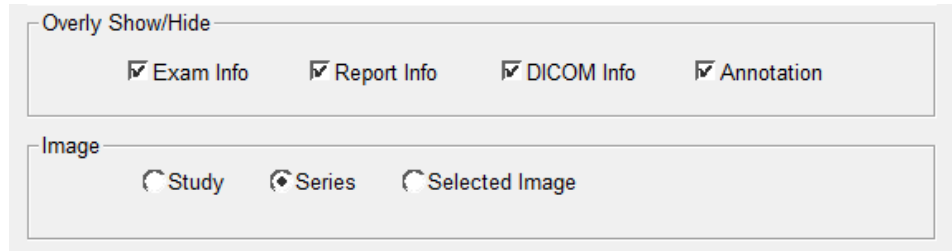

5. Click Print

6. In the print dialog box choose PDFCreator from the dropdown

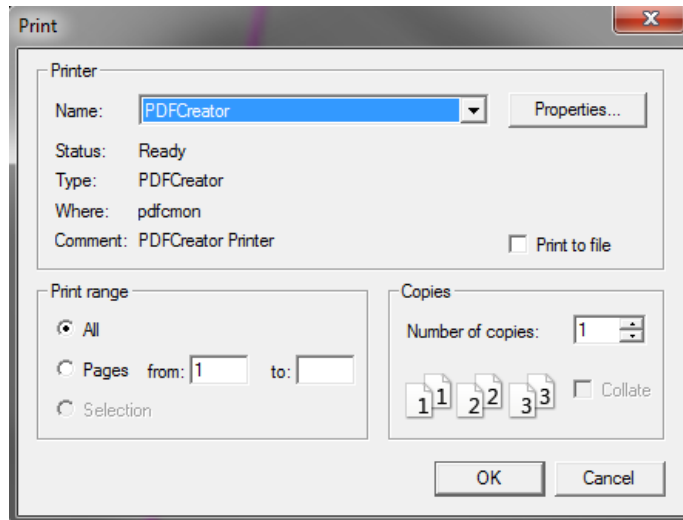

7. In the PDFCreator box name the file and click save

8. Choose a location to save the file

