

SR₃ ONE-PART POWDERED ***Rapid Developer Systems*** ***Cleaner***

(NON HAZARDOUS)

CONTENTS: 110 grams per sealed pouch

Directions:

Rinse tank & racks thoroughly with water.

Empty one pouch of cleaner into ½ gallon container and fill with warm-hot water.

Use cleaner as you would any liquid cleaner.

Best results is to let cleaner work in processor or on rack for 2-5 minutes...**NO LONGER** & then rinse thoroughly.

CAUTION: Acidic powder mixture.

Handle all chemicals safely; protect eyes, skin & clothing. Use with adequate ventilation and wash after handling.

Manufactured by:

**WHITE
MOUNTAIN
IMAGING**

46 Chenell Drive Concord, NH 03301 • (603) 228-5222 • FAX (603) 224-8159

SR₃ ONE-PART POWDERED
Rapid Developer Systems
Cleaner

(NON HAZARDOUS)

CONTENTS: 20 x 110 gram sealed pouches

Directions:

Each pouch will clean one large film processor Developer racks & system. See directions on each pouch.

CAUTION: Acidic powder mixture.

Handle all chemicals safely; protect eyes, skin & clothing. Use with adequate ventilation and wash after handling.

Manufactured by:

**WHITE
MOUNTAIN
IMAGING**

46 Chenell Drive Concord, NH 03301 • (603) 228-5222 • FAX (603) 224-8159